



**endoscopy
support
services, inc.**

®

E.S.S. ENDOSCOPE SAFEGUARD SYSTEM

INSTRUCTIONS AND INSTALLATION GUIDE for SS-1, SS-2 & SS-M ScopeSecure™ WALL MOUNT RACK and ISS-1 INSTRUMENT STORAGE SYSTEM

- 1) Choose a mounting area away from normal traffic patterns. For standard dry-wall mounting, please use all mounting hardware supplied with the rack.

NOTE: Construction laws of California and other U.S. States or other countries may require different mounting hardware than what is supplied. Please use your own judgment for hardware selection.

- 2) To aid in selecting the optimum height for mounting, insert the scope in the rack and hold it against the mounting surface at the desired location high enough so no part of the scope touches the floor (*does not apply to ISS-1 rack*).
- 3) Drill holes using a 1/4 inch drill-bit and insert plastic wall anchors into holes until only the collar is exposed (*wall anchors are not necessary if attaching to furring strip or stud*).
- 4) Place the screws through the finish washers and then through the rack mounting holes. Tighten screws firmly, but do not over-tighten.

The rack can be easily cleaned with warm, soapy water.

Do not use the following for cleaning:

- Alcohol
- Alcohol based solutions
- Harsh solvents
- Autoclave

**3 Fallsview Lane • Brewster, NY 10509
1-800-FIX-ENDO (800.349.3636) • Ph: 845-277-1700 • Fax: 845-277-7300
Website: www.endoscopy.com • E-mail: ess@endoscopy.com**