



endoscopy
support
services, inc.

®

ForcepSecure™

FORCEPS SAFEGUARD SYSTEM

INSTRUCTIONS AND INSTALLATION GUIDE FOR FS-1 ForcepSecure™ WALL MOUNT RACK

- 1) Choose a mounting area away from normal traffic patterns. Please utilize all mounting hardware supplied with the rack.
- 2) To aid in selection the optimum height for mounting, insert a set of forceps in the rack and hold it against the mounting surface at the desired location.

Make small pencil marks on the surface where the screw holes are located. These marks will show you where the screws will enter the mounting surface.

We suggest locating at least one mounting hole of a furring strip or stud if possible.

- 3) Drill ¼ inch holes through the marks drawn on the wall in **step 2**. **Do not drill through a furring strip or stud.** *The screws can securely bite into wood without the aid of a wall anchor.* Only when using wall anchors is it necessary to drill (for wall anchors, go to step 4. If not using wall anchors, skip to step 5).
- 4) Insert the wall anchors into the holes drilled through the wall until only the collar is exposed.
- 5) Place the screws through the finish washers and then through the rack mounting holes. Tighten screws firmly, but do not over-tighten.

The rack can be easily cleaned with warm, soapy water.

DO NOT USE:

- Alcohol
- Alcohol based solutions
- Harsh solvents
- Autoclave

for cleaning purposes.

3 Fallsview Lane • Brewster, NY 10509

1-800-FIX-ENDO (800.349.3636) • Ph: 845-277-1700 • Fax: 845-277-7300

Website: www.endoscopy.com • E-mail: ess@endoscopy.com