




**endoscopy
support
services, inc.**

®

best
Your source for everything endoscopy since 1989™

3 Fallsview Lane • Brewster, NY 10509
Ph: (845) 277-1700 • Fx: (845) 277-7300

Web: www.endoscopy.com

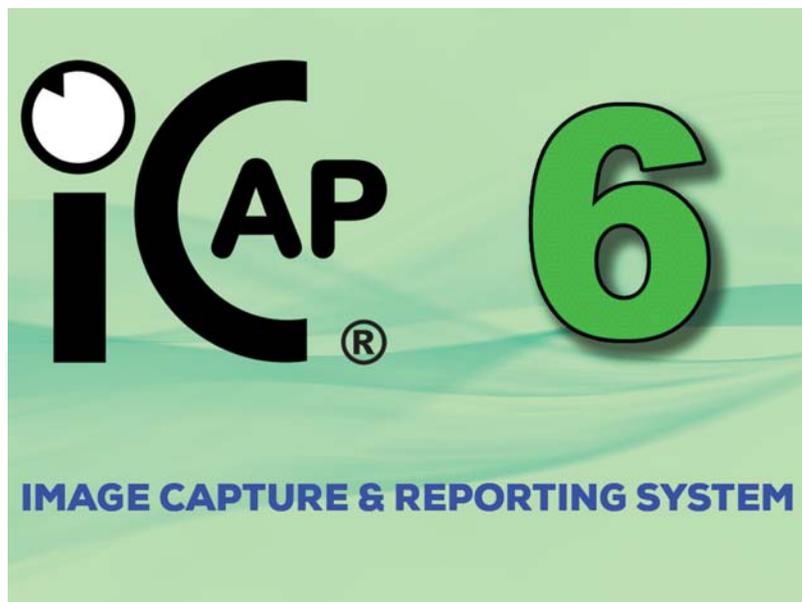

iCap® DICOM Compliant version of the Image Capture and Reporting System!

With iCap®, you can capture images and video directly onto a computer hard drive or a network location from any external video source.


The system catalogs the images by an identity for later review, reporting, e-mailing or printing. An image export feature allows you to place images directly into report templates. The iCap® system also gives you the ability to annotate your images with text, lines and shapes.

You can also record full-motion video clips of your procedures. Use the video clips for tutorials and seminars, send them to colleagues for analysis, or use them in educational environments such as college classrooms or labs.

With iCap, users can capture still images and video clips, then easily annotate images and catalog them by an identity for easy review later. The systems can send images directly to report templates and allows images to be easily integrated other programs like Power Point presentations and e-mail applications.

Because the software is Windows-based, it offers the user the ability to easily network multiple capture stations together with a server to create an image archival network. The images and reports are standard image files and documents that can easily be shared via email, FTP, and easily linked to most electronic record keeping systems and / or contact management programs.

Designed to work with or without a touch screen monitor, the iCap system can be used across a variety of settings. It utilizes an icon, push-button style interface, making the system as easy to use as an ATM machine. Pushing a button on the touch screen or using a mouse to click on the button achieves a desired result. Most actions can be completed with just one or two clicks.

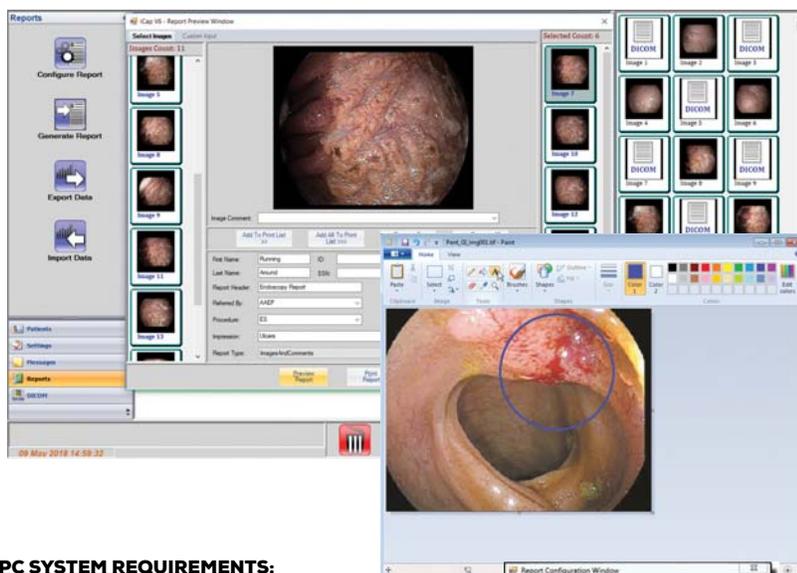

PC SYSTEM REQUIREMENTS:

- One (1) free USB port (for application security drive/dongle)
- An available USB port Camera or device that provides a IEEE-1394 or USB compatible digital format for recording
OR
A video digitizer that will convert analog video to digital.
- Microsoft Windows 7, 8 or 10
- DirectX 9.x drivers
- CD-ROM drive for software installation (download also available)
- 500MB of free hard disk space
- A dedicated color video graphics card is recommended for best results.
- Computer display/monitor




View online: www.endoscopy.com/iCap6D

**A Vendor Neutral Archive
made for modern medicine.**

Complete medical media management.

CloudEndo™

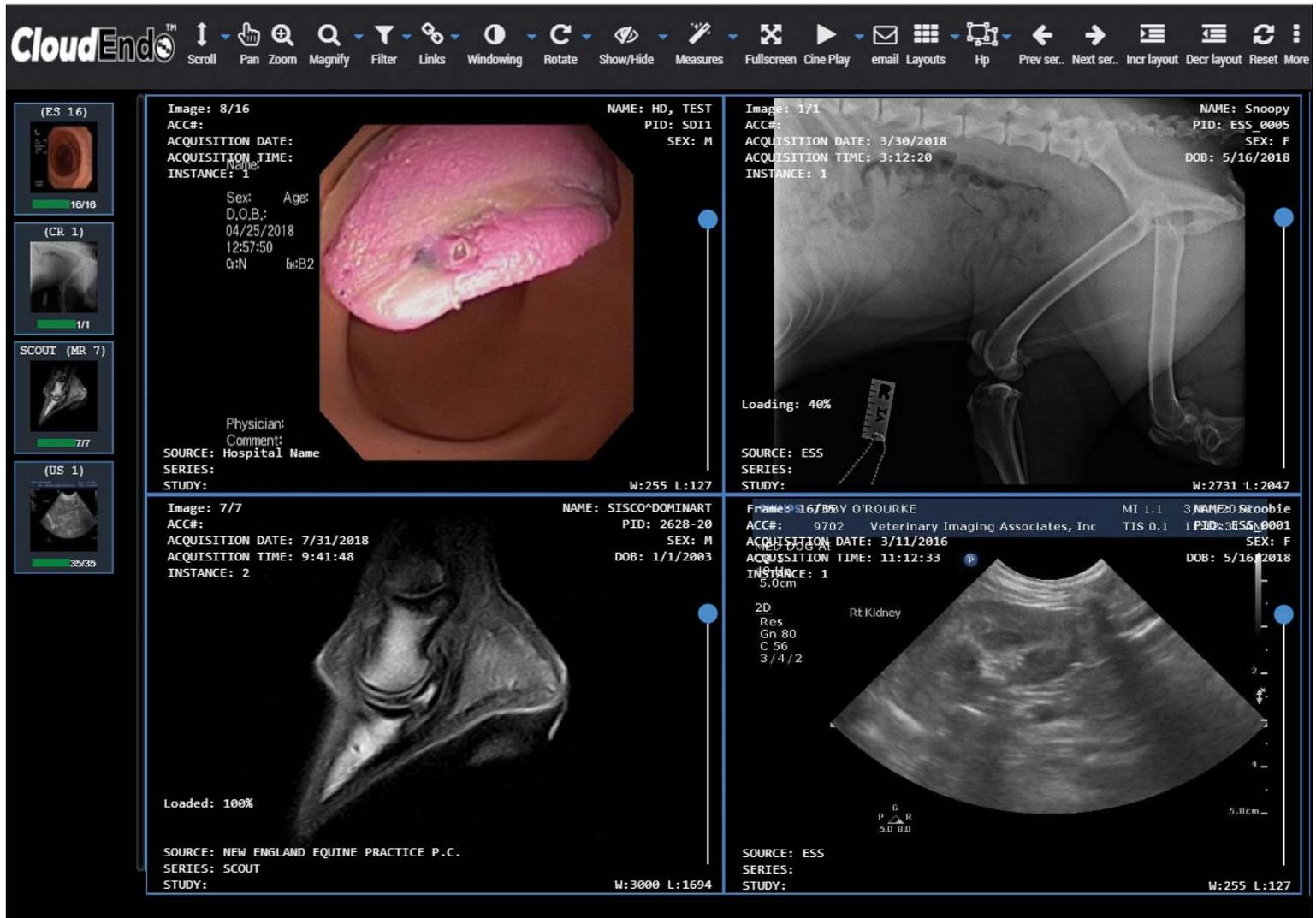

OUTSTANDING WORKFLOWS TO MEET UNIQUE NEEDS

Delivery of medical care creates a unique set of data and imaging demands for healthcare providers. Each department within the healthcare delivery system will generate large amounts of data. Each site can create multiple files and types of media from each of its modalities.

We understand the challenges healthcare networks face dealing with data management and modalities from various vendors.

CloudEndo.com was designed to manage all this data for you in our Vendor Neutral Archive that was made for you, because you are modern medicine.

Pairs perfectly with:

endoscopy.com/CloudEndo

www.dicom-services.com

